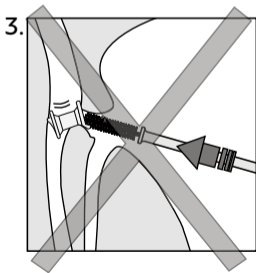
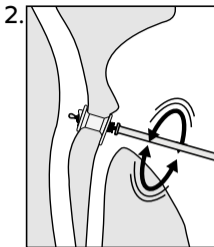
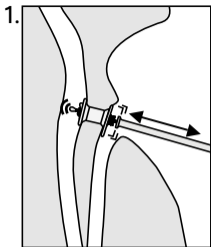
# PROVOX<sup>®</sup>

## ActiValve<sup>®</sup> Lubricant

## **Prescription information**

**CAUTION:** United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner.

The availability of this product without prescription outside the United States may vary from country to country.

## **Disclaimer**

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

## **Patents and trademarks**

Provox<sup>®</sup> ActiValve<sup>®</sup> is a registered trademark owned by Atos Medical AB, Sweden. For information about protective rights (e.g. patents), please refer to our web page [www.atosmedical.com/patents](http://www.atosmedical.com/patents)

Manufacturer; Hersteller; Fabrikant; Fabricant;  
Produttore; Fabricante; Fabricante; Tillverkare; Pro-  
ducent; Prodsent; Valmistaja; Framleiðandi; Tootja;  
Ražotājs; Gamintojas; Výrobce; Gyártó; Výrobca;  
Proizvajalec; Producent; Producător; Proizvođač;  
Proizvođač; Κατασκευαστής; Производител;  
Üretici; მწარმოებელი; Ἀρτοποιητής; İstehsalçı;  
Производитель; Produsen; Pengilang; 製造業者;  
제조사; 製造商; 制造商; היצרן; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productieda-  
tum; Date de fabrication; Data di fabbricazione; Fecha  
de fabricación; Data de fabrico; Tillverkningsdatum;  
Fremstillingsdato; Produksjonsdato; Valmistuspäi  
vä; Framleiðsludagsetning; Valmistamise kuupäev;  
Ražošanas datums; Pagaminimo data; Datum výroby;  
Gyártás időpontja; Dátum výroby; Datum proizvodn-  
je; Data produkcji; Data fabricației; Datum proizvodn-  
je; Datum proizvodnje; Ημερομηνία κατασκευής;

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## **Indications for use**

For use with Provox ActiValve only.

Lubricating the inner lumen of the Provox ActiValve prosthesis helps to prevent sticking of the valve that might otherwise occur e.g. after sleep.

## **Instructions for use**

Once a day, after cleaning in the morning, lubricate the inner lumen of the Provox ActiValve prosthesis with one drop of Provox ActiValve Lubricant using a different Provox Brush. See illustrations.

Do not put more than one drop on the brush. Avoid touching the brush with the ActiValve Lubricant bottle when dispensing a drop. Be patient when squeezing the lubricant bottle. The lubricant bottle releases the thick silicone oil very slowly. If you get Provox ActiValve Lubricant on your skin wash it off with soap and water.

After use clean the brush according to the Instructions for use that accompanies the Provox Brush.

The ActiValve Lubricant can be used until expiration date or 8 months after opening the bottle, whichever occurs first. Register the date of opening on the bottle using a water resistant permanent marking pen.

**Note:** Avoid touching the stomal mucosa with the brush during application of the Provox ActiValve Lubricant.

Incidental ingestion of small amounts of Provox ActiValve Lubricant is harmless. Incidental aspiration of small amounts of Provox ActiValve Lubricant is also harmless, but may cause coughing.

No health problems have been reported regarding repeated aspiration of small amounts of silicone oil, but it is still better to be cautious and avoid it. Use only one drop of Provox ActiValve Lubricant each time and make sure that any superfluous silicone oil is removed before using the brush (no dripping brush).

## **Disposal**

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

## **CAUTION:**

- Overuse of the lubricant in quantity or frequency can result in severe coughing, pneumonia or serious breathing problems.
- Do not apply the Provox ActiValve Lubricant directly from the bottle to the prosthesis since this could deposit the liquid in the lungs and cause severe coughing, pneumonia or serious breathing problems.
- Do not enlarge the bottle opening of the Provox ActiValve Lubricant.
- Keep the Provox ActiValve Lubricant out of reach of children and pets that may be harmed by inhaling or swallowing it.

## **Reporting**

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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