PROVOX[®]

Adhesive Remover

MD

CE

UK
CA

42 °C
108 °F
RT
2 °C
36 °F

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Ingredients/ Inhaltsstoffe/ Ingrediënten/ Contient/ Ingredienti/ Ingredientes/
Ingredientes/ Innehållsämnen/ Indholdsstoffer/ Innhold/ Ainesosat/
Innihaldsefni/ Koostis/ Sudétis/ Složení/ Összetevők/ Zloženie/ Sestavine/
Składniki/ Ingrediente/ Sastojci/ Sastojci/ Συστατικά/ Ингредиенты/
Съставки/ İçindekiler/ Ραφαρμολογικά/ Bahan-bahan/ 成分 성분 成分/
املكقيات / מרכיבים

Hexamethyldisiloxane, Isopropyl myristate

Contents

EN - ENGLISH.....	4
DE - DEUTSCH.....	5
NL - NEDERLANDS.....	6
FR - FRANÇAIS.....	8
IT - ITALIANO.....	9
ES - ESPAÑOL.....	10
PT - PORTUGUÊS.....	12
SV - SVENSKA.....	13
DA - DANSK.....	14
NO - NORSK.....	16
FI - SUOMI.....	17
IS - ÍSLENSKA.....	18
ET - EESTI.....	19
LT - LIETUVIŲ KALBA.....	21
CS - ČESKY.....	22
HU - MAGYAR.....	23
SK - SLOVENČINA.....	25
SL - SLOVENŠČINA.....	26
PL - POLSKI.....	28
RO - ROMÂNĂ.....	29
HR - HRVATSKI.....	30
SR - SRPSKI.....	32
EL - ΕΛΛΗΝΙΚΑ.....	33
RU - РУССКИЙ.....	34
BG - БЪЛГАРСКИ.....	36
TR - TÜRKÇE.....	37
HY - ՀԱՅԵՐԵՆ.....	39
MS - BAHASA MELAYU.....	40
JA - 日本語.....	42
KO - 한국어.....	43
ZHTW - 繁體中文.....	45
HE - עברית.....	46
AR - العربية.....	47

Instructions for use

Intended Use

Provox Adhesive Remover is a single use wipe to help laryngectomized patients remove Provox Adhesives and Provox Silicone Glue.

CONTRAINDICATIONS

Hypersensitive or breached skin or known allergies to ingredients.

Application

Stop the additional oxygen supply, if applied, and remove the HME before applying Adhesive Remover.

When using a Provox FlexiDerm, Provox XtraBase, or Provox StabiliBase, first apply the Provox Adhesive Remover on top of the adhesive. This is not necessary for the other adhesives. Then, for all Provox adhesives, grasp the finger lift tab and apply Provox Adhesive Remover on the edge and underneath the adhesive. Carefully remove the adhesive and adhesive residue from the skin. Clean the skin with water and soap or use Provox Cleaning Towel afterwards.

PRECAUTIONS

- Avoid deep inhalation of solvent during application and be careful that liquid does not drip into stoma.
- In case you experience signs of irritation or redness, discontinue use of product. If the irritation persists, consult your physician.

WARNINGS

- For external use only;
- Keep out of reach of children;
- Avoid contact with eyes. In the case of accidental contact, flush eyes well with water;
- Do not apply directly to open wounds and mucous membranes;
- Single patient use only. Re-use may cause cross contamination.
- Do not cut the tissue in pieces, small pieces may be aspirated.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.
Atos Medical UK Ltd Tottle Road
Atos Medical AB