PROVOX[®] LaryButton[™] Sizer Kit



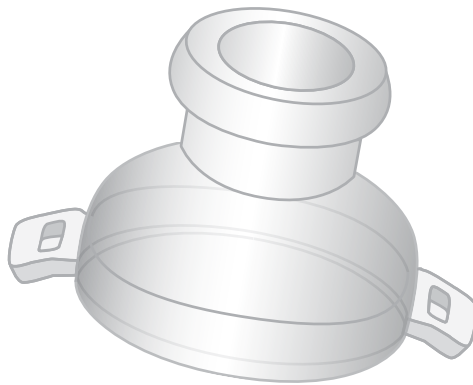
Figure 1.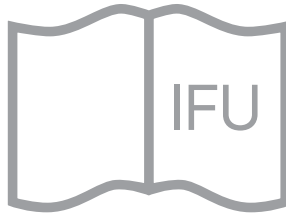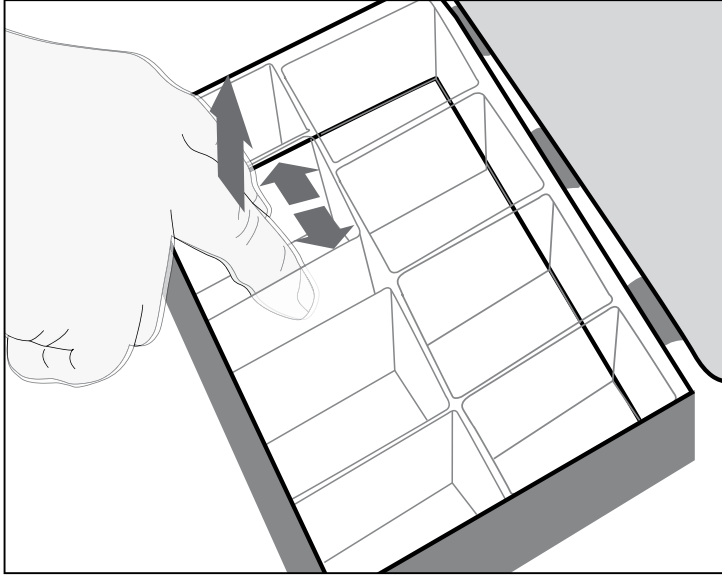

Prescription information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden and Provox® LaryButton™ is a trademark owned by Atos Medical AB, Sweden.

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore; Fabricante; Fabricante; Tillverkare; Producent; Produzent; Valmistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas; Výrobce; Gyártó; Výrobca; Proizvajalec; Producent; Producător; Proizvođač; Proizvođač; Κατασκευαστής; Производител; Üretici; მწარმოებელი; Ἀρτοποιητής; İstehsalçı; Производитель; Produzen; Pengilang; 製造業者; 제조사; 製造商; 製造商; היצרן; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productiedatum; Date de fabrication; Data di fabbricazione; Fecha de fabricación; Data de fabrico; Tillverkningsdatum; Fremstillingsdato; Produksjonsdato; Valmistuspäivä; Framleiðsludagsetning; Valmistamise kuupäev; Ražošanas datums; Pagaminimo data; Datum výroby; Gyártás időpontja; Dátum výroby; Datum proizvodnje; Data produkcji; Data fabricației; Datum proizvodnje; Datum proizvodnje; Ημερομηνία κατασκευής; Дата на производство; Üretim tarihi; წარმოების თარიღი; Ἀρτοποιηման ամսաթիվ; İstehsal tarixi; Дата изготовления; Tanggal produksi; Tarikh pembuatan; 製造日; 제조일자; 製造日期; 製造日期; תאריך ייצור; تاريخ التصنيع

Use-by date; Verfallsdatum; Houdbaarheidsdatum; Date limite d'utilisation; Data di scadenza; Fecha de caducidad; Data de validade; Används senast; Anvendes inden; Brukes innen; Viimeinen käyttöpäivä; Sijasti notkunardagur; Kõlblik kuni; Derīguma termiņš; Naudoti iki; Datum expirace; Felhasználhatóság dátuma; Dátum najneskoršej spotreby; Rok uporabnosti; Data ważności; Data expirării; Rok uporabe; Rok upotrebe; Ημερομηνία λήξης; Срок на годност; Son kullanim tarihi; ვარგისობა; Ἐπισημασμένη ἡμερομηνία; Son istifadə tarixi; Срок годности; Gunakan sebelum tanggal; Tarikh guna sebelum; 使用期限; 사용 기한; 使用截止日期; 使用截止日期; תאריך אחרון לשימוש; تاريخ انتهاء الصلاحية

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Intended use

The Provox® LaryButton™ Sizer Kit is intended for use by the prescribing clinician to determine the size(s) of LaryButton that should be prescribed to the patient.

The Sizer Kit should be used only by a prescribing clinician who has read the LaryButton Manual. A copy of that manual comes with the Sizer Kit. It can also be viewed on the Internet at www.atosmedical.com.

The Sizer LaryButtons are intended for the sizing procedure only. After the correct size(s) have been determined a new LaryButton(s) shall be prescribed to the patient for actual use. LaryButton order information is found on the last page of this manual.

Contraindications

The Sizer Kit in itself does not have specific contraindications.

Do not use the Provox LaryButton, or use it only with special care, in cases of tracheostoma tissue problems such as damaged mucous membrane, granulation tissue formation, and vulnerability with a higher tendency to bleed.

The Provox LaryButton may be contraindicated for patients with bleeding disorders or undergoing anticoagulant treatment.

Device description

The Sizer Kit is a box which contains samples (“Sizers”) of commercially available Provox LaryButtons. The sizes of these Sizers and actual Provox LaryButtons are the same and are indicated on the products themselves and in the bottom of the outer storage box.

Each Sizer in the Sizer Kit is stored in an individual removable polypropylene box. This makes it possible for the prescribing specialist to remove the individual storage boxes with the Sizers from the outer storage box individually. This allows for hygienic handling of both the Sizers and the storage boxes.

After each sizing session, the Sizer(s) with its individual storage box(es) must be cleaned, disinfected, dried and steam sterilized according to the accompanying “Instructions for cleaning and sterilization”. The outer storage box must also be cleaned if contaminated. The Sizer LaryButtons and their individual removable storage boxes are

thereafter put back at the appropriate position as indicated in the bottom of the outer storage box.

WARNINGS

- TO AVOID THE TRANSFER OF DISEASE, NEVER USE A CONTAMINATED SIZER LARYBUTTON.

After using a Sizer LaryButton, it becomes contaminated through handling and contact with the patient’s stoma. To avoid the transfer of disease from one patient to another, use appropriate hygienic procedures and do not put the LaryButton and its individual storage box back in the outer storage box after use. Make sure that each used Sizer and individual storage box is cleaned, disinfected, dried and steam sterilized. The outer storage box must also be cleaned if contaminated.

- The Sizer LaryButtons are intended for the sizing procedure only. After the correct size(es) have been determined, new LaryButton(s) shall be used by the patient.

PRECAUTIONS

- To reduce risks of traumatizing the tracheostoma during the Sizing procedure: Carefully read the LaryButton Manual before using the LaryButton Sizer Kit in order to understand the warnings, precautions and instructions for use of the LaryButton during insertion and replacement.
- The Sizers and the individual storage boxes are designed to withstand 10 cleaning and steam sterilization procedures at 137°C/278°F for 3 minutes or at 134°C/273°F for 18 minutes. Always ensure mechanical integrity before each use.
- The outer storage box is not designed to withstand steam sterilization.

Adverse events

The following are possible risks associated with the use of LaryButton Sizer Kit:

- Transfer of disease(s). In order to reduce the risk, use appropriate hygienic procedures. Clean, disinfect, dry and steam sterilize the Sizer LaryButton and individual storage box.
- Use of wrong size Sizer LaryButton during sizing procedure. To reduce the risk, make sure that the correct Sizer LaryButton is packaged in

its correct position as indicated in the bottom of the outer storage box.

Instructions for use

Checkout procedures

Do not use the LaryButton Sizer Kit if the package is damaged or the tamper seal has been opened.

Check that correct size(s), as listed with the product, are available in the Sizer Kit.

Make sure that the Sizer Kit has been cleaned and sterilized as appropriate.

Always ensure mechanical integrity of the LaryButton before each use.

Removing the Sizer from the outer storage box

First remove the individual storage box with the Sizer from the outer storage box without contaminating the remaining LaryButtons, individual storage boxes and outer storage box (Figure 1). Then remove the Sizer from its individual storage box.

Stoma shape and choosing the right Provox LaryButton

There is a wide variety in stoma shapes and diameters that are suitable for LaryButton use. Although optimal fit and performance of the Provox LaryButton may be achieved best in patients having a so-called “tracheostoma lip”, many more stomas may retain the LaryButton satisfactorily. In any case, the proper fit may result from trying out several sizes.

Diameter

The goal is to create a self-retaining, comfortable and airtight fit between the Provox LaryButton and the tracheostoma. To achieve an airtight fit and good retention (which is especially important for hands-free speech), the diameter of the tracheal end of the button should slightly exceed the diameter of the tracheostoma entrance. Measure the size of the tracheostoma (largest diameter) with a ruler and fit the Provox LaryButton accordingly.

Length

Care must be taken to choose a length that does not reach the tracheal flange of the voice prosthesis. See the chapter entitled Warnings in the Provox LaryButton manual. In most cases the shorter size

will provide the right fit, in which the tracheal end of the Provox LaryButton fits right behind the edge of the tracheostoma, without reaching the tracheal flange of the voice prosthesis. Obviously, in patients without a voice prosthesis these precautions concerning the length of the LaryButton are not relevant.

Cleaning and sterilization

For the Sizer LaryButtons and the individual storage boxes: Clean, disinfect and steam sterilize the Sizers and their individual storage boxes separately. Put the individual storage box upside down during cleaning in washer-disinfector and sterilization.

Read the “Instructions for cleaning and sterilization”. If this information is missing, please contact your supplier or visit our website on www.atosmedical.com.

The outer storage box has been tested to withstand the alkaline detergent; Suma Med Super LpH® (Johnson Diversey) and the disinfectant; Gigasept® Instru AF (Schülke&Mayr). Cleaning and disinfection of the separated outer storage box and size indicating sheet can be performed with this detergent and disinfectant according to the methods described in the “Instructions for cleaning and sterilization”.

Repacking after cleaning and sterilization

Make sure that the outer storage box is clean and dry before placing the reprocessed individual storage boxes and Sizers back in the outer storage box.

Check the Sizers for mechanical integrity before repacking.

Put the Sizers in their individual storage boxes in the correct location according to the size indicator in the bottom of the outer storage box. Make sure to use appropriate hygienic procedures to avoid contamination of the Sizers and individual storage boxes.

How supplied

The device is manufactured in a cleanroom and supplied non-sterile.

Device lifetime

The Sizer LaryButtons shall be replaced every 2 years, or after 10 cleaning and steam sterilization procedures, whichever comes first. The individual

storage boxes withstand 10 cleaning and steam sterilization procedures and cannot be replaced separately. Check mechanical integrity of the device prior to each use.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Ordering information

See last page of this manual.

Date of printing

See version number on the back cover of this manual.

User assistance information

For additional help or information please see back cover of this manual for contact information.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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Caution

United States Federal law restricts the sale, distribution and use by, or by order of a physician or a licensed practitioner of devices identified herein with the symbol "Rx".

The prescription-free availability of these products outside the United States may vary from country to country.

This page is deliberately left empty

This page is deliberately left empty

This page is deliberately left empty

This page is deliberately left empty

No. 90799, 2019-09-03

Atos
atosmedical.com

Atos Medical AB
Kraftgatan 8, P.O. Box 183, SE-242 22 Hörby, Sweden
Tel: +46 (0)415 198 00 • info@atosmedical.com