TRACOE experc Set vario
(REF 420, REF 422, REF 423)

TRACOE experc Set vario XL
(REF 421, REF 424, REF425)

DE	Gebrauchsanweisung
EN	Instructions for Use
FR	Instructions d'utilisation
IT	Istruzioni d'uso
ES	Instrucciones de uso
PT	Instruções de uso
DA	Brugsanvisning
FI	Käyttöohje
NO	Bruksanvisning
HU	Használati utasítás
SV	Bruksanvisning
NL	Gebruiksaanwijzing
HR	Upute za uporabu
SL	Navodila za uporabo
CS	Návod k použití
RO	Instrucțiuni
TR	Kullanma talimatı
RU	инструкция по применению
PL	Instrukcja używania
EL	Οδηγίες χρήσης
KO	사용 설명서
ZH	使用说明
HE	הוראות לשימוש
AR	دليل استعمال

TRACOE medical GmbH

Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany

Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200

info@tracoe.com, www.tracoe.com

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Note: Please read the instructions for use carefully. They are part of the described product and must be available at all times. For your own and your patients' safety, please follow instructions below.

The TRACOE *experc Set* consists of two components: The TRACOE *experc Dilation Set* and a TRACOE Tracheostomy Tube with minimally invasive insertion system. Please find necessary information on the general description, functional description, intended use, patient population, indications and technical description in their respective instructions for use.

1. Intended Use and Indications for Use

Intended Use: TRACOE *experc Dilation Set* for Percutaneous Tracheostomy is intended for performing a percutaneous dilation tracheotomy, applying the Ciaglia technique with a Seldinger guide wire. TRACOE *vario* tracheostomy tubes are indicated for providing tracheal access for airway management especially to those patients with unusual anatomy or patients with thick necks. It may be used for up to 29 days.

Indications: This medical device is indicated for cases where access to the respiratory tract is required by means of a percutaneous dilation tracheotomy, with subsequent placement of a tracheostomy tube.

2. General Description

A TRACOE *experc Set vario* consists of TRACOE *experc Dilation Set* for Percutaneous Tracheostomy (REF 520) and one of the following TRACOE *vario* Tracheostomy Tubes with minimally traumatic insertion system:

TRACOE *experc Sets vario*

REF 420: TRACOE *vario* Tracheostomy Tube, spiral-reinforced with cuff (REF 450-P)

REF 422: TRACOE *vario* extract Tracheostomy Tube, with subglottic suction line and cuff (REF 470-P)

REF 423: TRACOE *vario* Tracheostomy Tube, with cuff (REF 460-P)

TRACOE *experc Sets vario XL*

REF 421: TRACOE *vario XL* Tracheostomy Tube, spiral-reinforced with cuff (REF 451-P)

REF 424: TRACOE *vario XL* Tracheostomy Tube, with cuff (REF 461-P)

REF 425: TRACOE *vario XL* extract Tracheostomy Tube, with subglottic suction line and cuff (REF 471-P)

3. Procedure

1) Please read the Instructions for Use of both components (Dilation Set and tracheostomy tube) together before performing the tracheotomy.

2) Apply the TRACOE *experc Dilation Set* (REF 520) to create the dilative tracheostoma.

Follow with application of the included TRACOE *vario* Tracheostomy Tube.

4. Storage

Store the TRACOE products in their original packaging according to the conditions displayed on the packaging.

5. Packaging

The product is provided sterile (with ethylene oxide) which allows application under sterile conditions.

6. Disposal

Used products are to be disposed of in accordance with the valid national regulations, waste management plans, or clinical procedures governing biohazardous waste materials, e.g. the direct disposal in a tear- and moisture-resistant and secure bag or container, which is routed to the local waste disposal system for contaminated medical products.

For further recommendations, contact your hygiene officer in health facilities or the local waste management for homecare use.

7. General Terms and Conditions

The sale, delivery and return of all TRACOE products shall be affected exclusively on the basis of the valid General Terms and Conditions (GTC), which are available either from TRACOE medical GmbH or on our website at www.tracoe.com.

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